

# Collecting and sending hair samples to EAL



## 1. SAMPLE COLLECTION AND PRESERVATION

- Minimum of 0.2-0.4 g of hair - equivalent to one heaped tablespoon - in a clean zip lock plastic bag or envelope
- Hair should be clean, preferably washed with baby shampoo, well rinsed and dried. Cut small amounts of hair from the nape of the neck, close to the scalp
- Label bag with identification and date of sampling

## 2. COMPLETE EAL SAMPLE SUBMISSION FOR (SSF)

- Download SSF and Price List from the EAL website [www.scu.edu.au/eal](http://www.scu.edu.au/eal)
- Complete all relevant details on SSF
- Use test code SS-PACK-050 from the Price List
- Credit card and cheque are the preferred methods of payment

## 3. PACKAGE AND SEND SAMPLES

- Package sealed sample(s) in a postpak, suitable envelope or small box
- Enclose SSF in a sealed plastic sleeve along with any instructions
- Send or deliver to EAL - post and courier labels below



*(Results with guidelines will be supplied by email)*

### SEND BY MAIL:

TO: SOUTHERN CROSS UNIVERSITY  
Environmental Analysis Laboratory (EAL)  
PO BOX 157  
LISMORE NSW 2480

EAL Contact: 02 6620 3678 or  
Graham Lancaster: 0419 984 088

**NOTE : URGENT DELIVERY  
OF SAMPLES FOR TESTING**

### SEND BY COURIER:

TO: SOUTHERN CROSS UNIVERSITY  
Environmental Analysis Laboratory (EAL)  
Ground Floor, N Block  
Military Road, EAST LISMORE NSW 2480

EAL Contact: 02 6620 3678 or  
Graham Lancaster: 0419 984 088

**NOTE : URGENT DELIVERY  
OF SAMPLES FOR TESTING**

SCU7483